**PUSH MED KNEE BRACE****INTRODUCTION**

Push Med products provide solutions in the case of a disorder or reduced functioning of the locomotor apparatus.

Your new knee brace is a Push Med product, characterised by a high level of effectiveness, comfort and quality. The Push med knee brace increases knee stability and helps control pain. This increases physical strength. Mechanical support, adjustable compression and multiple features that keep the brace in place are the foundation of its performance.

Push med products are innovative and are manufactured for you with great care.

This ensures maximum security and greater freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push knee brace. If you have any questions, please consult your retailer or send an e-mail to [info@push.eu](mailto:info@push.eu).

**PRODUKTEIGENSCHAFTEN**

Your Push med knee brace offers a high level of excellent support and protection to the knee joint.

The knee joint is supported in the medial-lateral direction by innovative leaf spring joints hinges and static straps. This allows for unobstructed flexion and extension of the knee. The hinges follow the natural bending point of the knee.

The silicone application stimulates the proprioception of the knee joint, to adequately anticipate any imminent trauma.

So-called "comfort zones" of padded material have been created around vulnerable areas of the knee. A proper anatomical fit and the silicone inlays contribute to keep the brace in place during movement. Its small volume enables the brace to be worn underneath clothing without being noticed.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside soft and keeps the skin dry. The Push med knee brace can be washed normally.

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There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly at first.

Pain is always a warning sign. If the pain persists or symptoms worsen, it is advisable to consult your doctor or therapist.

**DETERMINATION SIZE / FITTING**

The knee brace is available in 5 sizes. Sit with your knee bent at 90°. Then measure the circumference around the narrowest part of your leg below the knee joint. The chart on the packaging shows the correct brace size for your knee.

A. Slide the brace over the knee joint. To do this, hold the brace by the plastic tabs above the hinges and pull it up the leg. Ensure that all the straps are loose. Fit it firmly.

B. Close the straps at the back and the front of the thigh. Varying the length at the back and the front allows you to optimally position the stiffeners on the side of the knee. The one-sidedness of the brace is balanced.

C. Close the straps that crisscross at the front of the shin.

**WASHING INSTRUCTIONS**

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions.

If you intend not to use the brace for some time, it is recommended that you wash and dry the brace before storage.

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used; after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

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Push® is a registered trademark of Nea International bv

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For details of the distributor in your country, refer to [www.push.eu/distributors](http://www.push.eu/distributors)

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