PUSH MED ANKLE BRACE

INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus. Approved by physicians and scientists.

Ihre neue Ankelbrance ist ein Push med Produkt, charakterised by a high level of effectiveness, confort and quality. The Push med ankle brace increases ankle stability and helps to control pain. It increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med ankle brace. If you have any questions, please consult your reseller or send an e-mail to info@push-brace.com.

PRODUCT CHARACTERISTICS

Your Push med ankle brace offers a high level of support and protection during the treatment or prevention of ankle injuries. It can be used in a wide range of cases and conditions. The combination of a zip and Velcro fastening ensures that the sock can be fitted easily. Non-elastic bands can be used to set the pronation and supination limits, without any impediments for walking. The elastic bands with integrated heel-lock, according to the functional tape concept, offer individually adjustable compression around the ankle. The silicone application on the inside of the bracket stimulates the ankle's proprioception. This is done to anticipate an imminent trauma.

The ankle brace has a perfect fit and can be worn in practically any shoe. „Comfort zones“ of padded material have been created around vulnerable areas such as the instep and the heel.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med ankle brace is easy to apply and can be washed without any problem.

INDICATIONS

- Moderate (residual) instability (under moderate stress)
- Treatment of acute lateral ankle ligament lesions (alternative to taping)
- Treatment of stable ankle fractures (Weber type A with isolated fracture of the lateral malleolus)
- Follow-up treatment of ankle fractures (incl. Jones fracture)
- Postoperative treatment of ligament injuries or fractures
- Capular irritation and tendinitis following ankle fracture or ligament lesions
- Arthrosis with instability
- Secondary prevention of ankle ligament lesions (under moderate stress with accumulation of fluid)
- Football ankle
- Impingement (capsular clamping; anterior and posterior)

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The ankle brace is available in 5 sizes. Position the foot at an angle of 90° in relation to the lower leg. Then measure the circumference around the heel and the instep of the foot, as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your foot.

- Slide the base around your foot, making sure it fits snugly. Keep the foot at an angle of 90° in relation to the lower leg. Then close the zip and the elastic fastening at the front.
- Close the non-elastic bands crosswise across the instep. The correct fitting sequence is determined by the increasing molecule symbols on the Velcro.
- The elastic bands are wound around the foot in the shape of the figure 8. Start by tensioning the heel-lock. Position the band diagonally across the instep in the direction of the inner foot.

Then pull the band underneath the foot towards the outside and diagonally across the instep to the inside of the lower leg.
- Close the elastic band with a single wrap around the lower leg.
- Tension the heel-lock of the second band. Position the band diagonally across the instep in the direction of the outer foot.
- Then pull the band underneath the foot towards the inside and diagonally across the instep to the outside of the lower leg.

Close the elastic band with a single wrap around the lower leg.

WASHING INSTRUCTIONS

Before washing, close the Velcro to protect the brace and other laundry. The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used; after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier). To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly. Always consult your physician or therapist if complaints persist or deteriorate.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

Push® is a registered trademark of Nea International bv, P.O. Box 1558, 6201 BN Maastricht, Netherlands www.push-brace.com

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©2039-2022 / 01