PUSH MED ELBOW BRACE EPI

INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus. Approved by physicians and scientists.

Your new elbow brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med elbow brace Epi guarantees improved functioning and pain relief. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med elbow brace Epi. If you have any questions, please consult your reseller or send an e-mail to info@push-brace.com.

PRODUCT CHARACTERISTICS

Your Push med elbow brace Epi offers local pressure on the extensors (tennis elbow) or the flexors (golfer's arm) of the wrist. This relieves the strain on the insertion of these muscles at the elbow joint. The brace can easily be fitted and closed around the lower arm with one hand. The bracing element around the arm ensures a proper distribution of pressure. The pad consists of a skin-compatible and elastic polymer. A proper anatomic fit and the material of the pad contribute to the brace remaining in place during movement.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med elbow brace Epi can be washed normally.

INDICATIONS

- Tennis elbow (lateral epicondylitis)
- Golfer's elbow (medial epicondylitis)

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The elbow brace Epi is available in 1 size. The measurements range from 20 to 35 centimetres.

Make sure that the brace's pressure point (at the level of the logo) is placed a few centimetres beside the painful insertion of the muscle towards the hand.

Tension the adjustable band and close it.

WASHING INSTRUCTIONS

- Before washing, close the Velcro to protect the brace and other laundry.
- The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.
- A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly.
- Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

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