PUSH MED KNEE BRACE

INTRODUCTION

Push med products provide solutions in the case of a disorder or reduced functioning of the locomotor apparatus. Approved by physicians and scientists.

Your new knee brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med knee brace increases knee stability and helps to control pain. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med knee brace. If you have any questions, please consult your reseller or send an e-mail to info@push-brace.com.

PRODUCT CHARACTERISTICS

Your Push med knee brace offers a high level of support and protection to the knee joint.

The knee joint is supported in the medio-lateral direction by means of a mechanical construction consisting of leaf spring joints and non-elastic connections. This does not obstruct flexion or extension of the knee. The stiffer hinges follow the natural pivoting point of the knee.

The silicone application stimulates the proprioception of the knee joint, to adequately anticipate any imminent trauma.

So-called comfort zones of padded material have been created around vulnerable areas of the knee. A proper anatomic fit and the material and the silicone application contribute to the brace remaining in place during walking. Its small volume enables the brace to be worn underneath clothing without being noticed.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med knee brace can be washed normally.

INDICATIONS

- Arthrosis with hydrops
- Rheumatoid arthritis with synovitis
- Functional instability after collateral ligament trauma
- Residual instability after collateral ligament trauma
- Late follow-up treatment after ligament surgery

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.


Pain is always a sign of warning. If the pain persists or complaints deteriorate, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The knee brace is available in 5 sizes. Place the thigh at an angle of 90° in relation to the lower leg. Then measure the circumference around the narrowest part below the knee joint. The table on the packaging shows the correct brace size for your knee.

- Slide the brace over the knee joint. To do this, hold the brace by the plastic stiffeners and pull it up the leg. Ensure that all the bands are loose. Ensure that the ring fits round the patella properly.
- Close the bands at the back and front of the thigh. The variation in the length allows you to position the stiffeners so that the brace fits snugly and works perfectly.
- Then close the overlapping bands around the leg.

WASHING INSTRUCTIONS

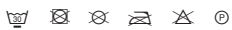Before washing, close the Velcro to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to refer the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- This brace contains rigid elements. If you are in doubt about its suitability in your sport, please contact your governing body.

Push® is a registered trademark of NEA International bv.

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