## PUSH MED SHOULDER BRACE PLUS

### INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus.

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### PRODUCT CHARACTERISTICS

Your Push med shoulder brace Plus consists of four components. The breast band, which is closed around the chest or upper waist, constitutes the basis of the brace. This band only needs to be adjusted once for the correct size. The 'quick lock' (A) can be used to open and close the breast band and to adjust it for maximum comfort. To prevent abduction of the shoulder as much as possible, the upper arm band fixes the upper arm against the body. The comfortable wrist/hand support keeps the arm fixed and supports it. The Push med shoulder brace Plus adds comfort through the shoulder band that has been added to the brace: this band provides additional immobilisation and support to the affected shoulder joint and allows the hand to be kept at the desired height. Each fixation point is individually adjustable, for an optimal fit in all circumstances. The brace is made of Sympress™, a comfortable and high-quality material. The microfibre used makes the inside soft to the touch and keeps the skin dry. The edges of the bands also feel soft. The Push med shoulder brace can be washed normally.

### INDICATIONS

- Collarbone fracture
- Status after (sub)luxation
- Status after operation of the shoulder joint
- Humeroscapular periarthritis
- Subcapital humerus fracture

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Make sure not to apply the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

### FITTING / SETTING

The shoulder brace is available in 3 sizes. For proper fitting, the chest circumference is measured 2 centimetres above the level of the hanging elbow. The table on the packaging shows the correct brace size for your chest circumference. The correct fitting sequence is determined by the increasing molecule symbols on the Velcro (B, C, D).

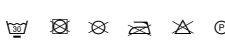
All Velcro bands marked with symbols must be opened; the others remain closed.

- Grasp the brace with the nonaffected arm at the wide section of the shoulder band.
- Put the affected arm and your head through the opening.
- Close the upper arm band by passing the band through the clasp and closing it just above the elbow (B).

*For the professional who supplies the brace: Before initial use, two bands of the brace must be adjusted for the patient once:*

- Adjust the shoulder band at the back to ensure that the breast band is at the same height as the upper arm
- Then adjust the breast band so that it fits closely to the body and feels comfortable for the patient.
- Close the breast band so that it fits comfortably (C).
- Legen Sie die Hand in das Handgelenkband und verschließen Sie dieses am Schulterband (A).
- Put your hand in the wrist/hand support and close it on the shoulder band (B).
- You may slide the wrist/hand support on the breast band (left/right) or pull it (up/down), to achieve the best possible fit.

### CARE AND MAINTENANCE

The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents. Before washing, close the Velcro to protect the brace and other laundry.

A gentle spin-dry cycle can be used. Then let the brace dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

### IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

Push® is a registered trademark of Nea International bv  
P.O. Box 305, 6199 ZN Maastricht-Airport, Netherlands  
[www.push.eu](http://www.push.eu)

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