## PUSH MED WRIST BRACE SPLINT

### INTRODUCTION

Push med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus. Approved by physicians and scientists.

Your new wrist brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push Splint guarantees improved stability and pain relief. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med wrist brace splint. If you have any questions, please consult your reseller or send an e-mail to [info@push-brace.com](mailto:info@push-brace.com).

### PRODUCT CHARACTERISTICS

Your Push med wrist brace splint offers a high level of support, protection and relaxation of the wrist, without restricting the functionality of the hand. The brace is easily fitted with one hand around the lower arm, the wrist and the hand. A circular, non-elastic closure ensures that the adjustable splint tightly fits the anatomy of the wrist. This restricts palmary flexion and places the hand in a functional position. So-called comfort zones of padded material have been created around vulnerable areas of the wrist. A proper anatomic fit contributes to the brace remaining in position during movements.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibrines makes the inside feel soft and keeps the skin dry. The Push med wrist brace splint can be washed normally.

### INDICATIONS

- Therapy-resistant strain complaints of the wrist extensors
- Moderate to serious rheumatoid complaints
- Carpal tunnel syndrome
- RSI

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

### DETERMINING SIZE / FITTING

The wrist brace is available in 4 sizes. Measure the circumference of the wrist, as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your wrist size.

- The correct fitting sequence is determined by the increasing molecule symbols on the Velcro.
  - Slide the base around the hand until it reaches the thumb.
  - Use the non-elastic band to set the required compression around the wrist.
  - Close the brace around the lower arm and hand.

### ADJUSTING THE SPOON

The spoon is made from Thermolyne®, a synthetic material that can be shaped both in a cold and in a warm state. This makes it easy to achieve a functionally optimal position of the wrist for each individual.

Using a hammer to tap the spoon into the desired shape, is the easiest solution. To shape the spoon in a warm state, the material must be heated to approximately 160°C, either in an oven or with a hair-drier.

### WASHING INSTRUCTIONS

Before washing, close the Velcro to protect the brace and other laundry. Remove the splint from the brace before washing. The brace can be washed at temperatures up to 30°C, on a delicate fabric cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

### IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist. Please be aware that an ill-fitting brace does not function properly.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to repeat the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

Push® is an registered trademark of Nea International bv, P.O. Box 1558, 6201 BN Maastricht, Netherlands [www.push-brace.com](http://www.push-brace.com)

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