ENGLISH

## PUSH MED WRIST BRACE

### INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus. Approved by physicians and scientists.

Your new wrist brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med wrist brace guarantees improved stability and pain relief. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med wrist brace. If you have any questions, please consult your reseller or send an e-mail to [info@push-brace.com](mailto:info@push-brace.com).

### PRODUCT CHARACTERISTICS

Your Push med wrist brace offers a high level of support, protection and rest of the wrist, without restricting the functionality of the hand. The anatomically shaped stiffeners limit palmary and dorsal flexion without obstructing it completely. The level of compression can be optimised by an easy-to-apply elastic band. The brace is easily fitted with one hand around the lower arm.

So-called comfort zones of padded material have been created around vulnerable areas of the wrist. A proper anatomic contribute to the brace remaining in position during movements.

The Bandage is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med wrist brace can be washed normally.

### INDICATIONS

- Residual instability
- Moderate to serious distortions
- Strain complaints in the wrist
- Therapy-resistant strain complaints of the wrist extensors
- Tendinitis of the wrist extensors
- Wrist fractures without dislocations
- Follow-up treatment of wrist fractures
- Arthrosis with hydrops
- RSI

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

### DETERMINING SIZE / FITTING

The wrist brace is available in 4 sizes. Measure the circumference of the wrist, as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your wrist size.

- The correct fitting sequence is determined by the increasing molecule symbols on the Velcro.  
Slide the base around the hand until it reaches the thumb.  
Close the brace around the lower arm.
- Use the elastic band to set the correct measurement range around the hand.
- Wrap the band around the wrist once, to apply the required compression to the wrist and close it at the top.

### WASHING INSTRUCTIONS

- Before washing, close the Velcro to protect the brace and other laundry.
- The brace can be washed at temperatures up to 30 °C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.
- A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

### IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

Push® is a registered trademark of Nea International bv, P.O. Box 1558, 6201 BN Maastricht, Netherlands [www.push-brace.com](http://www.push-brace.com)

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